


Adjusting Occlusion

How to Lengthen Anteriors Without Increasing Vertical Dimension

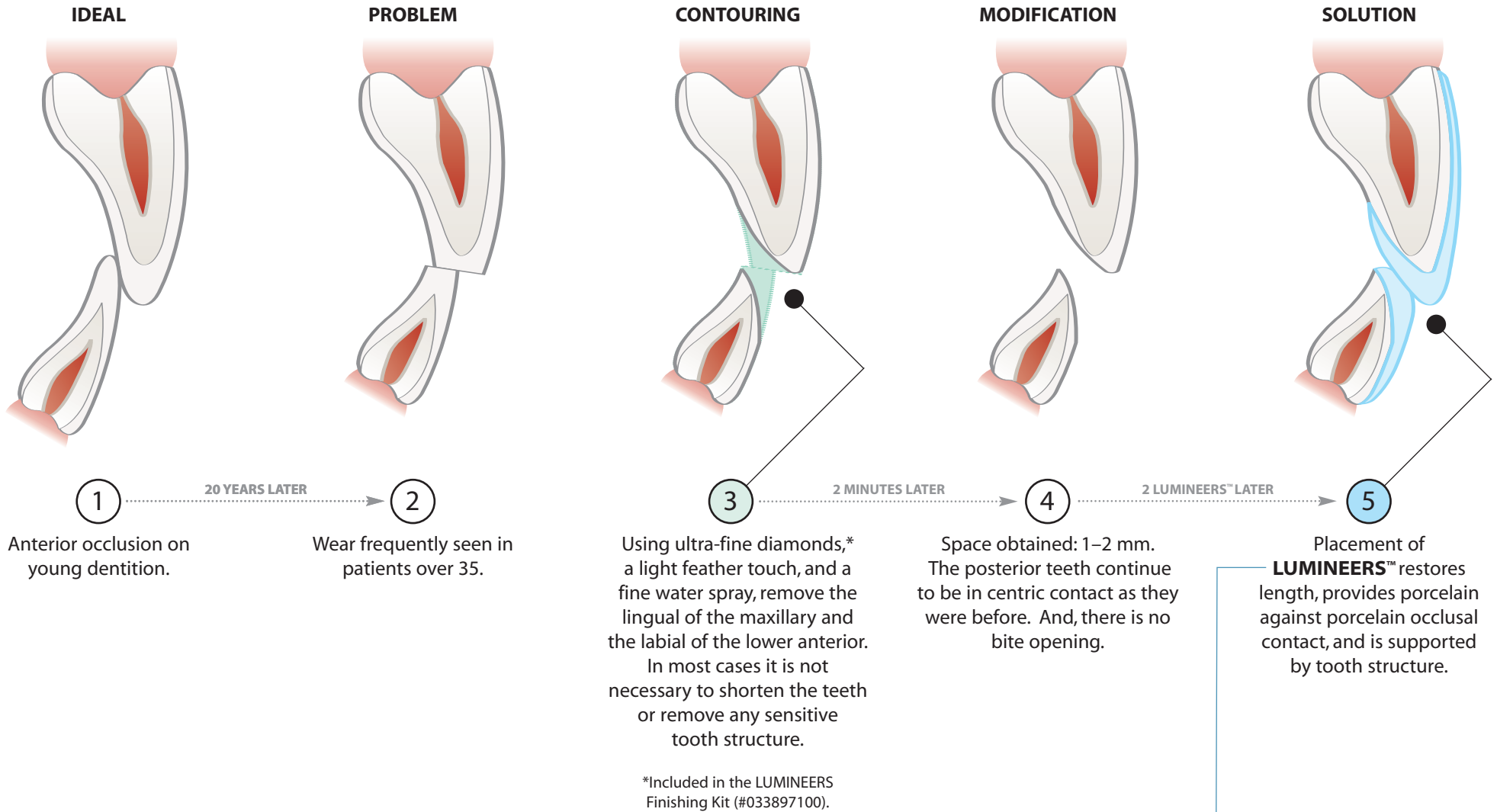

TIPS

- Create round edges on the maxillary teeth when the porcelain covers the incisal edges.
- When in doubt, send in models and photos for consultation to Cerinate Smile Design Studios. Call 800-872-8384 for Cerinate prescription forms and boxes.

DenMat®

LUMINEERS™
BY CERINATE®

How to Lengthen Teeth Without Increasing Vertical Dimension

—TWO CASE EXAMPLES

CASE 1



A. Contour guidelines



B. Lengthened teeth




C. Bite did not change



D. Finished result

CASE 2: Lengthening Cross Bite

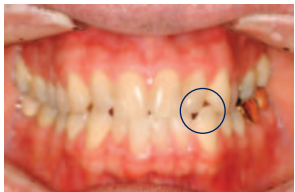

A. Before



B. Cosmetic contour




C. After contour




D. Finished result

See how thin or thick Cerinate LUMINEERS can be.

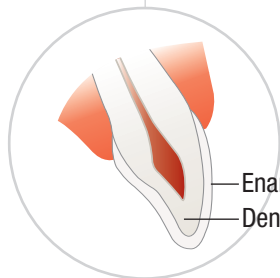

NORMAL TOOTH STRUCTURE

Enamel
Dentin

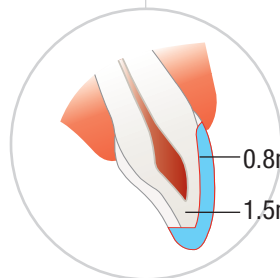


TRADITIONAL PREP VENEERS

Approximately 0.8mm–2mm healthy tooth structure removal

0.8mm
1.5mm–2mm


or

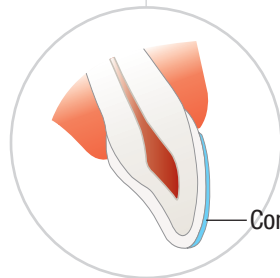

CONTACT-LENS-THIN LUMINEERS

No sensitive tooth removal
0.3mm–0.5mm added to the enamel

Contact-Lens-Thin

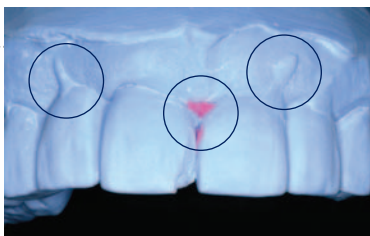

IMPERFECT MODEL EXAMPLE

Pour the model *before* you send the case in to make sure you have an accurate impression that provides sharp margins.



The recommended impression material is LUMINEERS™ PVS.

Kit No. 031389910

Adjusting Shades —Avoid Remakes!

Prevent remakes and reappointments by using the Cerinate Shade Modification Kit. With a palette of colors you can simply modify and opaque dark shades right at the chair. There is no need to remake the shade.



Mask tetracycline stains!



Cerinate Shade Modification Kit
No. 033909900



LUMINEERS™
BY CERINATE®